

# dr. detox

## The Ionic Sponge Oven Cleaner!

### Benefits:

- Eliminates greening, discoloration, and bubbling in porcelain restorations.
- Extends the muffle and the thermocouple life because there is no longer a need to take the muffle to high temperatures when cleaning the oven as with old-fashioned carbon products.
- Maintains the thermocouple accuracy by removing the ions of various metals that stick to the thermocouple and distort temperatures.



### Instructions:

1. Use no vacuum!
2. Use only one DR. DETOX FORMULA (material) at a time.
3. Fill provided crucible near the top with DR. DETOX FORMULA 1 or DR. DETOX FORMULA 2 material.
4. Place the crucible on the oven table.
5. Be sure the oven temperature is below 400° C / 800° F.
6. Pre-heat the oven for 3 minutes by increasing the temperature at the rate of 40° C / 65° F per minute to a high temperature of 700° C / 1300° F.
7. Hold the oven temperature at 700° C / 1300° F for 5 minutes.
8. Cool the oven for 3 minutes.
9. Discard DR. DETOX material after each use.

### Formula Removes:

#### FORMULA 1

Silver Ag  
Palladium Pd  
Copper Cu  
Cobalt Co

#### FORMULA 2

Aluminum Al  
Barium Ba  
Beryllium Be  
Calcium Ca  
Carbon C  
Cadmium Cd  
Cesium Cs  
Chromium Cr

Iron Fe  
Lithium Li  
Magnesium Mg  
Mercury Hg  
Potassium K  
Rubidium Rb  
Sodium Na  
Strontium Sr  
Zinc Zn

*Gold is not a problem alloy.  
It is other alloys in the  
gold that cause problems.*