

# wondergloss

## Model Shine!

Gives dental stones and plasters  
a professional high shine luster!

### Benefits:

- Provides dental stones and plasters with a high shine luster for that professional quality look.
- Prevents staining and chipping of models.
- Works on all dental stones and plasters.
- Only adds less than (1) micron of thickness to the stone or plaster so accuracy and quality are maintained.

*Wondergloss is not a dip or a model soap.*



# wondergel

\$12.95

Holds crowns, veneers,  
and small castings in place!

Request  
a Free  
Sample!

### Benefits:

- It's Like Having an Extra Helping Hand
- Easier to Handle Small and Loose Fitting Restorations
- No Risk of Contamination
- Water-Soluble

### Wondergel Instructions:

1 Place a small amount  
of Wondergel inside  
the restoration.

Restoration is ready to build-up  
or contour. (It is not recommended  
to remove gel prior to firing.)



2 Place restoration on die.  
(If excess gel is squeezed out  
at the marginal area, hold restoration with the finger and  
use air pressure to blow excess down and away from the margin.)

4 Easily remove  
restoration from die.

