

# wonderadmix®

## Make Perfect Models with the Wonderadmix Bubble Buster!



### Benefits:

- Increases PSI
- Improves hydration
- Creates a smoother stone
- Increases strength, hardness & density
- Prevents chipping & fractures
- Controls shrinkage & expansion
- Causes greater accuracy & fine margins
- Eliminates the need for surfactants

Request  
a Free  
Sample!

**\$39.<sup>95</sup>** 1.5 gl Kit (5-25cc Syringes) - 200 uses

**\$69.<sup>95</sup>** 3.0 gl Kit (10-25cc Syringes) - 400 uses

**\$64.<sup>95</sup>** 3.0 gl Refill Kit (10-25 cc syringes/no bottle)

### Wonderadmix Sample Instructions:

\*You'll need 1 empty 32 fl. oz. (quart) (946 mL) bottle with a lid. Empty 20 mL (1 sample syringe) of concentrated Wonderadmix into the bottle. Fill the bottle with 32 fl. oz. of water. SHAKE WELL (shake before each use)! Use this mixture of Wonderadmix and water in place of the recommended amount of water as directed by the stone/plaster manufacturers. \*If a quart bottle is not available, you may use a 16 oz. bottle (pint) (473 mL) and add 10 mL (1/3 oz.) or one-half of the sample syringe of Wonderadmix. Do not use with automated dispensing equipment.