

Request
a Free
Sample!



\$46.95

wonderfill®

Tongue & Void Filler

The pre-mixed EZ in, EZ out Tongue and Void Filler that saves time and produces a sleeker, presentation-worthy model. Eliminates time spent mixing alginate filler or grinding out stone from lower impressions. Fills in bubbles, air pockets, missing teeth, and undercuts to enhance the fit of appliances and bleaching trays. Excellent in creating a temporary dam when making partial molds. Also, holds up under heat when making heat-shrink retainers. Water-soluble, requires no clean-up, and sold in a 1.13 kg resealable container. When used with Wonderform Casting Ring or the Wonderortho Formers, Wonderfill saves time and material costs.

www.wonderfill.com
+1.254.772.4661

No Mixing
No Mess **No Grinding**

Wonderfill® Instructions



Remove a palmful of non-toxic Wonderfill from its handy tub.



Form a domed bridge in the tongue area.



With Wonderfill in the tongue area, pour plaster/stone.



After plaster/stone fully sets, pop out non-sticky Wonderfill.



Using only water & a brush, remove the slight residue that might remain.