



Products

WonderFill

- Introduction
- The Story
- How it Works
- Compare
- Free Sample

WonderForm

- Product Details
- How it Works

WonderOrtho

- Product Details
- How it Works

WonderAdmix

- Product Details
- Free Sample

WonderPeg

- Product Details
- How it Works
- Use It For . . .
- Free Sample

wonderform

The innovative and EZ way to simplify the box and pour process.
Produce better molds in less time.

The box and pour process has never been so easy.

The WonderForm simplified application eliminates all mixing of materials, eliminates all wax products, and dramatically reduces the time to prepare cast models. Research shows that using the innovative WonderForm process saves an average of 25 minutes per model.



BENEFITS INCLUDE:

**No Mixing, No Mess,
No Wax Products**

Works with ALL Laboratory,
Orthodontic and General
Dentistry Applications

**Works with ANY Casting
Material, Stone, Silicone, and
Rubber-Based products**

Results in a Superior Quality
Mold while Taking Less than
Five Minutes to Complete

254.772.4661

WonderFill

- Introduction
- The Story
- How it Works
- Compare
- Free Sample

WonderForm

- Product Details
- [How it Works](#)

WonderOrtho

- Product Details
- How it Works

WonderAdmix

- Product Details
- Free Sample

WonderPeg

- Product Details
- How it Works
- Use It For . . .
- Free Sample

No mixing. No mess. No wax products needed.

Make superior quality molds in under 5 minutes.

Remove approximately 10 ounces of the pre-mixed WonderFill and flatten out about one-half inch thick into an approximate 5 inch diameter base pad on a 5" x 5" plastic or glass base plate.



Insert tray with impression. The impression material can be of any type of product on the market today. Firmly press down the tray with impression material into the WonderFill base pad.



The WonderForm casting ring is positioned, similar to that of a cookie-cutter, and then firmly pressed down. There is no need to feather the outer perimeter with wax beading or heated wax.



Peel away the excess material from around the outside perimeter of the WonderForm casting ring. The excess can be used with the next casting.



Move the base plate to the vibrator and pour in the liquid cast material, any cast material can be used. Any casting stone, siliconē, and rubber-based products. The WonderForm casting ring does not split open while the cast material cures.



When the casting material is cured, release the retaining clip on the WonderForm casting ring and remove the finished mold. This revolutionary technique developed for the box & pour process of casting dental models has proven results that can be used with ALL laboratory, orthodontic, and general dentistry applications.



Materials Needed:

- Preliminary Impression
- WonderFill
- WonderForm Casting Ring/Model Base Former
- 5" x 5" Plastic or Glass Base Plate
- ANY Standard Casting Material