

wonderfreeze

Multi-purpose cooling spray!

Benefits:

- Cooling vacuum forming products
- Cooling orthodontic wire
- Removing sticky wax
- Making crowns
- Making & setting teeth

\$19.95

16.95 each
for a case of 12

Wonderfreeze Instructions

- **Vacuum Forming Materials:** Hold the can in the upright position with the nozzle directly next to the vacuum formed processed material. Pull the trigger to coat the processed material thoroughly. Processed material can be removed immediately.
- **Denture Teeth, Large Amounts of Wax, and Metal:** Hold the can in the upright position with the nozzle 6 to 12 inches from the wax. Pull the trigger and spray directly on area to be cooled. For denture teeth, this will hold them in place. For large wax areas and metal finishing, this will quickly cool the case.
- **Crown Work:** Wait about 5 minutes until the metal is no longer red in color. Hold the can in the upright position with the nozzle about 12 to 14 inches from the full gold crown just soldered. Pull the trigger and spray the full gold crown and investment (if used) until cooled. This will harden the metal. Do not spray early for this can weaken the crown.
- **Chilling Memory Type Arch Wire:** Pull the trigger and spray on a saturated cotton swab tip and apply to the wire. For best results, wrap the wire with a piece of gauze and saturate it with Wonderfreeze. The gauze should not be dripping prior to application.



www.wonderfill.com
+1.254.772.4661