wondergloss

*Gives dental stones and plasters
a professional high shine luster!*

Wondergloss is used to provide dental stones and plasters with a high shine luster for that professional quality look when presenting models. It prevents staining and chipping of models. It works on all dental stones and plasters from orthodontic models to the practical models in the dental laboratory and dental office. Wondergloss only adds less than (1) micron of thickness to the stone or plaster so accuracy and quality are maintained.

DIRECTIONS FOR USE:

1. Make sure the model is completely dry before applying Wondergloss.
2. Place any identification needed on the models before application (case numbers, names, etc.). This will seal them on the stone or plaster making them smudge free with a quality and shiny appearance.
3. Evenly apply 1 to 3 coats with a brush depending on the amount of gloss desired.
4. Wondergloss will dry quickly and models will be ready for use.

KEEP OUT OF REACH OF CHILDREN DO NOT MIX WITH OTHER CHEMICALS

CAUTION: May cause eye & skin irritation.

FIRST AID: Eyes- In case of contact, immediately flush eyes with plenty of water for at least 15 minutes. Remove contact lenses. If irritation persists, contact a physician. External - Flood with water. Internal - Give large quantities of water.

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