


wonderpeg

Instant Firing Support!

New and Improved!

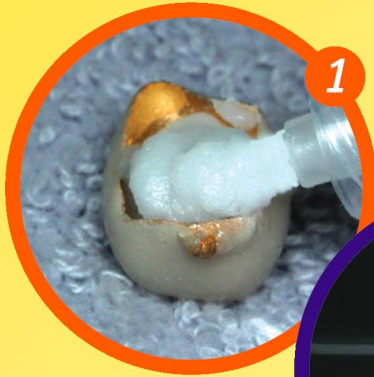

\$6.50
(5 min)

Request a Free Sample!



Benefits:

- Use with any type of restoration or porcelain
- No firing distortion or contamination
- No drying time required
- Stable up to sustained temperature of 1300° C (2375° F)
- Stabilization when using pegs, posts, or pins
- Use for stabilization when soldering
- Use for creating custom firing pegs
- Use for repairing holes in castings with or in place of platinum foil
- Use for a temporary heat shield for temperatures up to 1925° C (3500° F)

Instructions:

- Using the Wonderpeg syringe, simply fill the crown or jacket; no drying time is necessary.
- The crown or jacket is ready to be fired using a ceramic firing peg. It is easily balanced on the ceramic peg and does not tilt during the firing. Wonderpeg offers zero firing distortion for temperatures from 400° C to 1300° C.
- Simply remove the crown with a slight twist from the ceramic peg after firing and tap off any residual Wonderpeg material.

www.wonderfill.com
+1.254.772.4661